

ANTENNA INSTRUCTIONS

(Model 902A, 902AL)




Avery Biomedical Devices (ABD) warrants each new antenna against defects in materials or workmanship for a 90 day period from date of receipt.



Antennas should be examined frequently and discarded if there are signs of usage defects such as torn insulation or green coloration indicating conductor corrosion.



ABD recommends that antennas be replaced every six months.

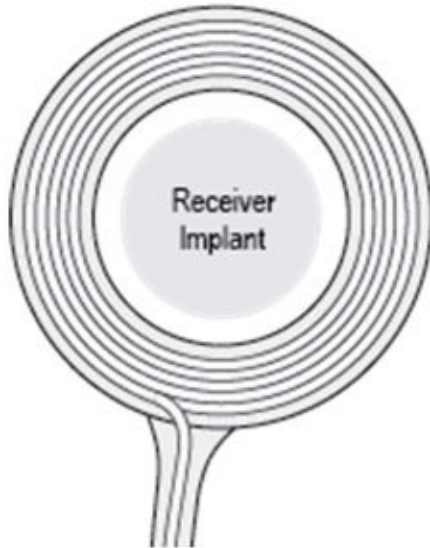

To Couple the Antenna with the Implanted Receiver:



- Position the antenna loop directly over the receiver implant as shown so it completely surrounds the receiver
- This position must be maintained for optimum receiver signal reception. If the loop is not directly over the receiver, power is lost to the implanted system.

To Disconnect the Antenna to the Transmitter:

- Hold the transmitter firmly.
- Grasp antenna connector by metal barrel NOT by the rubber covering.
- Pull straight UP and OUT. DO NOT rotate, twist or force to disconnect.

To Connect the Antenna to the Transmitter:

- Hold the connector between the thumb and forefinger.
- Locate, then align the flat surfaces of the "D" shaped guides on both the antenna connector and the transmitter socket.
- Press antenna connector into transmitter socket until it "clicks" into place.

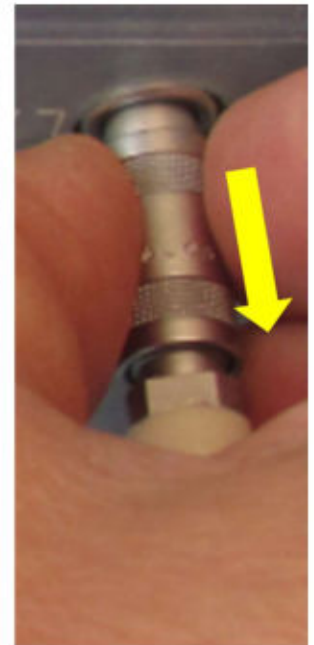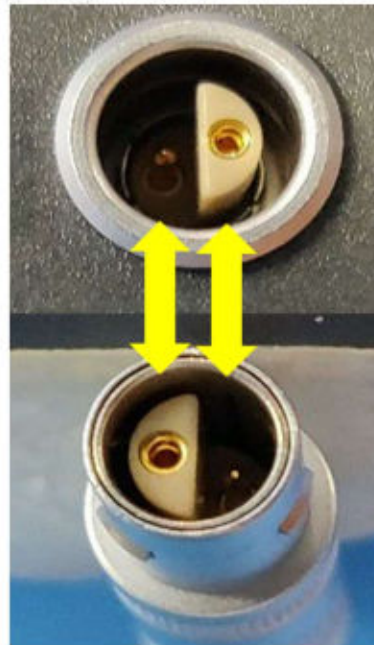

Care and Maintenance:

Proper care and regular cleaning will preserve the antenna and add to antenna life.


- Wash the antenna periodically with soap. Rinse and dry thoroughly.
- Periodically remove adhesive tape building with cloth dampened with rubbing alcohol. (DO NOT SOAK)

The following should NOT be done:

- DO NOT pull or jerk antenna to adjust antenna position.
- DO NOT wrap cord around transmitter body.
- DO NOT tie knots in cord.
- DO NOT stick pins through the cord.
- DO NOT allow the metal connector to become wet.



Advena Ltd.
Tower Business Centre, 2nd Flr,
Tower Street, Swatar, BKR 4013, Malta
SRN: MT-AR-000000234

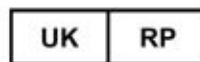

Advena Ltd. UK
Pure Offices, Plato Close
Warwick, CV34 6WE
United Kingdom

